D Gebrauchsinformation

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GB Instructions for use

DC composite core is a self-curing, radiopaque core composite, which is automatically mixed and applied directly into the cavity. The excellent flow properties of DC composite core guarantee optimal adaptation to the tooth structure. Even awkward areas are easy to access using the intraoral tips.

Indication

1. All types of core build-up
2. Cementing root canal posts

Cartridge handling

The material is dispensed and mixed automatically by extruding through a mixing tip. Remove the transport seal before using a new cartridge. Then attach the mixing tip and, if required, an intra oral tip. When using for the first time, discard the material initially extruded from the mixing tip (about the size of a pea), to secure an optimal mixture. Following application, the mixing tip should be used as a seal and should be left on the cartridge until the next application. The mixing tip should only be replaced with a new one when the cartridge is to be used again.

Cavity preparation

Use of a rubber dam is recommended. Remove caries and existing fillings before applying DC composite core and ensure suitable pulp/ dentine protection. Thoroughly rinse the cavity and dry with air. Avoid both excessive residual moisture and overdrying of the dentine. Do not use purely light-curing, one-component adhesives for bonding DC composite core. A chemically curing or dual-curing two-component adhesive should be used for adhesive retention of DC composite core to the tooth structure. Adhere to the manufacturer's instructions for use.

Recommended application

1. Core build-ups

To facilitate application of DC composite core, place a matrix on the tooth to be prepared. DC composite core can then be applied directly into the preparation. The material can be contoured using commercially available instruments, e.g. DC Heidemann spatula. The matrix should only be removed after the DC composite core has fully cured. When using prefabricated crowns, apply DC composite core directly into the crown. Then place the crown on the prepared tooth and hold in position until the DC composite core is fully cured.

The working time of DC composite core is 1 min. The restoration can be fully loaded again after a minimum of 4 minutes. DC composite core is automatically fully cured after this time.

2. Cementation of root canal posts

Condition the root post according to the manufacturer's instructions for use. After preparing the root canal, etch the inside of the canal according to the bonder manufacturer's instructions and apply the bonder. If the bonding system used has light-curing components, remove any excess in the inside of the canal with a paper point before light curing to avoid blocking the canal cavity.

Apply DC composite core to the prepared canal. The root post can also be coated with Composite core build-up prior to fitting. After the root post has been fitted, DC composite core can also be used for the core of the coronal section of the tooth. A matrix should be used when building up the core due to the low viscosity of the material.

Fabricating a temporary restoration

If an acrylic temporary restoration is to be fabricated on the prepared tooth, it is essential to use a suitable separating agent (e.g. Vaseline) to ensure separation.

Working times

- | | |
|-------|----------------------------------|
| 0 min | Application into the preparation |
| 1 min | End of the working time |
| 4 min | DC composite core is fully cured |

Please note

- Do not use DC composite core together with materials that contain eugenol.
- Do not use DC composite core in case of allergies to components of the material or the rare case of contact allergies are known.
- Avoid skin contact with the paste. On accidental contact, wash with soap and water. On contact with the eyes, rinse immediately with plenty of water and consult a doctor.

Side effects

There are no known systemic side effects. In individual cases a hypersensitivity to components of the material cannot be ruled out.

Composition

Barium glass, Bis-GMA-based resin matrix, pyrogenic silica, activa-tor, catalyst (benzoyl peroxide), stabilizer and additive. Filler content: 72% by weight = 49% by vol. (0.02-4 µm).

Classification

DC composite core complies with the requirements of ISO 4049:2000, type 2, class 1.

Storage

Don't store above 25 °C / 77 °F. It is recommended storing opened cartridge in refrigerator. Do not use after date of expiry.

Available product sizes

DC composite core

REF 702340	1 Cartridge (A3) à 25 ml
	35 Mixing tips
	35 Intra oral tips

Keep away from children! For dental use only!