Gebrauchsinformation

1-Schicht-Adhäsiv

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Single Layer Adhesive

Description

DC Adhesive is a light-curing single-component adhesive indicated for the direct adhesive restorative technique with composites and compomers. DC Adhesive requires only one application and is quick and easy to use. DC Adhesive is technique tolerant with regard to different moisture conditions of the dentin surface. It can consequently be used on both moist dentin (wet bonding technique) and dry dentin.

Composition

DC Adhesive contains methacrylate monomers, initiators, stabilizers and organic solvent in an aqueous solution.

Indications

DC Adhesive is used as an adhesive for direct light-curing composite and compomer restorations.

Contraindications

DC Adhesive should not be used if the patient is known to be allergic to any of the ingredients listed or if the prescribed working technique cannot be used.

Side effects

DC Adhesive may cause a sensitizing reaction in patients with a hypersensitivity to any of its ingredients. In these cases, the material should no longer be used.

Interactions

Materials containing eugenol or clove oil inhibit the polymerization of DC Adhesive. These types of materials should not be used in conjunction with DC Adhesive.

Technique

Preparation / Lining

- Prepare cavity according to the principles of adhesive restoration.
- Remove caries. Clean non-prepared dentin with a cleaning paste free of oil and fluoride. Then rinse thoroughly and dry.
- Use a base or liner if required. Apply the base or liner only onto deep areas of the preparation. Do not coat the remaining cavity walls. Use these surfaces for surface bonding with DC Adhesive.

Conditioning of enamel and dentin

- Apply DC Ätzel (a 37% phosphoric acid etchant) to the enamel and dentin, beginning with the enamel margins. Wait for 15 seconds.
- Thoroughly rinse off the phosphoric acid gel with water. Dry the dentin surface with air free of water and oil. The dentin should not become too dry. A slightly moist dentin surface enhances the performance of DC Adhesive (wet bonding technique).

DC Adhesive can, however, also be used when the dentin has been desiccated inadvertently due to its water base.

Application of DC Adhesive

- Apply DC Adhesive onto the conditioned tooth structure with a brush. With a light motion brush the material gently into the dentin for 10 sec.
- Wait for 15 seconds. Remove excess material with air free of water and oil.
- Apply a thin layer (1 to max. 2 mm) of composite or compomer restorative (e.g. DC Feinsthybrid).
- Light-cure according to the manufacturer's instructions for use (e.g. for 40 secs. when using DC Feinsthybrid).
- Proceed with the usual layering technique and process materials according to the manufacturer's instructions for use.

Notes

Total Etch Technique

DC Adhesive is used in conjunction with the Total Etch Technique. With this procedure, dentin and enamel are conditioned simultaneously with phosphoric acid. Etching causes decalcification, exposing collagen fibres.

- It is usually sufficient to etch the enamel and dentin for 15 secs. Primary tooth enamel and heavily fluoridated enamel may require an extended etching time.

However, do not etch the dentin for more than 15 secs., as this may result in excessive decalcification.

Drying the dentin

The dentin needs to contain a certain degree of moisture to assure optimum adhesion (wet bonding principle). After conditioning, the dentin should be dried with an air syringe until the surface is slightly glossy. However, no visible excess water should remain on the tooth surface. Excessive drying usually compromises the bonding mechanism.

DC Adhesive solvent technology and drying procedure

DC Adhesive can be used on moist dentin surfaces in accordance with the wet bonding technique. Nevertheless, DC Adhesive also tolerates dry surfaces and surfaces that have been desiccated inadvertently. The aqueous components of the solvent rehydrate the dentin, restoring the physiological water content of the dentin.

Application of DC Adhesive and removal of excess material

DC Adhesive should be applied with a brushing motion for 10 sec. During the waiting time of 15 sec., the adhesive is diffusing into the dentin. Excess amounts of DC Adhesive should be blown off until a thin layer is attained. After removing the excess material, a glossy DC Adhesive film should be visible on the tooth substance.

Lining

If a liner or base is used in the given clinical situation, apply it only to deep areas of the preparation. Do not coat the remaining cavity walls. Use these surfaces for bonding with DC Adhesive.

Isolating

For restorative purposes, we recommend using a rubber dam whenever possible. This provides a clear, clean and dry operating field. It is also favourable and safe for patients and dental professionals alike.

Light-curing

DC Adhesive is a light-curing material. It should be applied in combination with light-curing restorative materials. DC Adhesive can be light-cured together with the initial composite or compomer layer. According to the user's preference, DC Adhesive can alternatively be light-cured before applying the restorative materials. DC Adhesive is not capable of self-curing and will not bond to self-curing composites. DC Adhesive should only be used in cases where polymerization light can reach the adhesive.

Warning

DC Adhesive is irritant and inflammable. It contains organic acids and organic solvent. Contact of the material with the skin, mucous membrane and eyes may cause irritations and must be avoided. If DC Adhesive comes in contact with the eyes, rinse immediately with copious amounts of water.

If required, consult a physician. If the material comes in contact with the skin, rinse with water and soap.

Storage

- Shelf life: see the date of expiration on the label/packaging.
- Do not use DC Adhesive after the date of expiration.
- Close bottle immediately after use. Exposure to light will cause premature polymerization.
- Store (2 - 28 °C / 36 - 82 °F)

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The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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DC Dental Central GmbH