
DC Amalgampistole

DC Amalgam gun
DC Pistolet à Amalgame
DC Pistola para amalgama
DC Amalgampistol

DC DentalCentral GmbH • Owienfeldstr. 6 • D-30559 Hannover • Germany • www.dental-central.de • E-Mail: office@dental-central.de

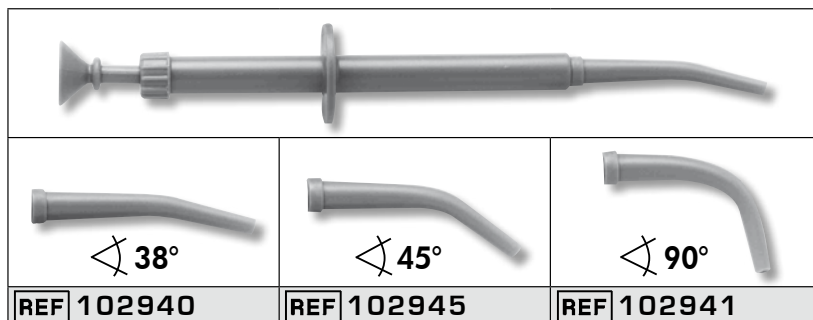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GB Instructions for use

Important! Please follow these instructions strictly:

- The amalgam gun is an amalgam carrier, not a condenser, please dose the forces accordingly.
- Always expell the amalgam immediately.
- After each use take the amalgam gun apart and clean it.
- Sterilize always assembled.

How to clean the amalgam gun:

- When holding the tip of the amalgam gun, first turn the little screw, and then the middle part. (never turn the tip, as this could damage the plunger)
- Clean all parts and assemble in reverse order as in step 1 (please note that the shape of the tip must correspond to the one in the plunger)
- Disinfect (with a solution of your chosing) and /or sterilise the amalgam gun in the autoclave or in dry heat up to 134° C.

By following these instructions, the high-tech material ensures that the amalgam gun will never clogg. The precision manufacturing avoids infiltration of amalgam between plunger and tip.

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How to clean the amalgam gun:

1. When holding the tip of the amalgam gun, first turn the little screw, and then the middle part. (never turn the tip, as this could damage the plunger)
2. Clean all parts and assemble in reverse order as in step 1 (please note that the shape of the tip must correspond to the one in the plunger)
3. Disinfect (with a solution of your chosing) and /or sterilise the amalgam gun in the autoclave or in dry heat up to 134° C.

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