DC Comp-Polierer

DC Dental Central Großhandelsges. mbH • Carl-Zeiss-Str. 2 • D-22946 Trittau • www.dental-central.de

Stand 07/2009

D Gebrauchsinformation

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Stand 07/2009

GB Instruction for use

Non-compliance with the maximum allowable RPMs will lead to an elevated safety risk. Insufficient water cooling may injure the teeth and the contiguous tissue irreversibly.

INSTRUCTIONS

Workplace

Remove surface contamination with a paper towel.

Storage and transport

No specific requirements. It is recommended to process the instrument as soon as possible after using it.

Cleaning preparations

See disinfection and sterilisation. Disassembly is not necessary.

Cleaning: Manual

Equipment: cleaning agent (e. g. DC Instrumentendbad), brush, running water.

Procedure:

1. Wash off surface contamination on the instrument (temp. < 30° C)
2. Apply cleaning solution with a brush on all surfaces.
Use a suitable brush for cleaning cannulizations and blind holes so that all places will be reached.
3. Hold the instrument under running water for 3 minutes.
The running water must rinse out the cannulizations, and the blind holes must be filled with water and emptied several times.

Cleaning: Automatic

Equipment: cleaning/disinfection machine, cleaning agent (e.g. DC Instrumentenbad)

Procedure:

1. Load the instruments into the machine.
2. Set on standard cycle, wash for at least 5 minutes and rinse for 3 minutes.
3. When removing the instruments, check cannulizations, blind holes, etc. for visible contamination. If necessary, repeat cycle or clean manually.

Disinfection

Disinfectant, can be used according to the instructions on the label. When cleaning automatically, the equipment can be rinsed at the end for 10 minutes at a temperature of 80° C.

Maintenance

Identify and replace blunt or damaged equipment.

Checking and performance test

All instruments: visual inspection for damage and wear and tear. Corners and edges should not show notches and should be even. Check long and narrow instruments (particularly rotation instruments) for distortions. If instruments belong to a larger system, it should be checked together with the corresponding elements.

RPM instructions

The maximum allowable RPMs must not be exceeded. The recommended and maximum allowable RPMs vary from product to product. To check the recommended and maximum allowable RPMs please consult our current catalogues or the packaging. If the maximum allowable RPMs are exceeded, this may cause vibrations, which can destroy the polisher, bend the shaft and/or break the instrument, i.e. it may lead to an increased safety risk for users, patients or third parties. Compliance with the recommended speed range will lead to the best work results.

Limits to processing

Frequent processing impacts the instruments only slightly. The end of the product lifecycle is normally determined by wear and tear and/or damage through use.

Sterilisation

Vacuum autoclave, at least 20 minutes at 134° C, or according to RKI-directive, not exceeding 150° C. The products are not suitable for sterilisation in chemiklave or a hot air disinfectant.

Storage

No specific requirements.

Additional information

When sterilising several instruments in an autoclave, the indicated maximum load prescribed by the manufacturer may not be exceeded.

Contacting the manufacturer

DC Dental Central Großhandelsges. mbH
Carl-Zeiss-Str. 2
D-22946 Trittau
Tel.: +49 (0) 4154 / 84 37 - 0
Fax: +49 (0) 4154 / 84 37 - 33
E-Mail: office@dental-central.de

The manufacturer has confirmed that the above-mentioned instructions are suitable for preparing the re-use of an instrument. The person processing the instrument bears the responsibility that the actual processing with the used equipment, materials and employees in the processing installation will produce the desired results. Normally, validations and routine monitoring of the process are necessary for that. In addition, each deviation from these instructions by the processing person should be evaluated diligently in terms of efficiency and potential negative consequences.