DC Carbon Post / DC Fibre Post

DC DentalCentral GmbH • Owiedenfeldstr. 6
D-30559 Hannover • Germany • www.dental-central.de

D Gebrauchsinformation

Präparation des Stiftbettes:

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GB Instruction for use

Preparation of post space:

- 1 Using the appropriate Turbine, prepare the root canal with the correct canal expander. It is recommended to use a rubber dam. Make sure to have sterilized the drill tips by 134° C /273° F in an autoclave.
- 2 Make sure to check the X-Ray and diameter of the root opening before choosing the post and matching drill. The selection of the appropriate post and drill is the responsibility of the user.
- 3 Insert the post 6-7 mm into the root canal. The post should be able to be inserted without extreme pressure or twisting. A test fit may be necessary to check the bite angle.
- 4 Mark the appropriate length (occlusion length), and cut the post extraorally with a sharp diamond instrument. Make sure to keep dust to a minimum. (moisten the post before cutting). Do not use a clipper to cut the post! This will damage the material composition, and the structure of the material!
- 5 After the mechanical preparation of the root canal, clean and disinfect the root canal. Rinse the root canal with a disinfectant solution (i.e. H₂O₂). Dry with paper points and then blow dry.
- 6 Handle the post with an instrument – avoid contact with hands. Clean the post with alcohol and dry. When using DC Fibre Post silanize the post (follow manufacturer's instructions). Using a root canal filler (Lentulo), fill the root canal with a thin dual-hardening composite. (Make sure to follow the directions and preparation steps from products from other manufacturers).
- 7 Apply the composite on the root post. With slow rotation, apply the composite in the root canal with a lentulo. With slow rotation, apply the post with the post sleeve and wait for the composite to set, so that the post stays in place. After removing the excess material, light cure the material. (1000 mW/cm² für 40s). Make sure to follow manufacturer's instructions.
- 8 After curing, fill the stump with thick flowable composite. (Make sure to follow the manufacturer's instructions for the composite). It may be necessary to use matrix bands to form the composite properly.
- 9 Use diamond coated instruments and a turbine with water spray to contour the composite.

Cleaning and Disinfection

The DC Reamers are not sterile upon delivery. Clean and sterilize them in an autoclave (134° C /273° F). Make sure to use a cleaner that is intended for cleaning rotary instruments. (We recommend DC Bohrerbad NF). Do not clean the drills in a thermo disinfectant. Store the drills where they stay dry and dust free. Check for defects before each use. Damaged or dull drills should be no longer used and discarded.

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