# DC Copalvarnish

DC Dental Central · Carl-Zeiss-Straße 2 · D-22946 Trittau

REF 702215

D

## Gebrauchsinformation

### Produktinformation

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## Instructions for use

### Product information

DC Copalvarnish is an adhesive varnish for sealing the dentin before fitting inlays, gold crowns, jacket crowns or bridge attachments, and for fixing restoration materials, as well as for sealing of glassionomer-cements and amalgam fillings.

### Application field

DC Copalvarnish is used for straight-forward surface sealing of the dentin (cavities and stumps), of glassionomer-cements (e.g. DC Glas-IonomerCem) and amalgam fillings, as well as for fixing of sealing materials.

### Application

After preparation, clean the surfaces to be treated and dry thoroughly with warm air. Put DC Copalvarnish onto a suitable applicator a drop at a time and apply quickly, dry for 30 seconds with warm air and repeat application. Once the dentine takes on a lustre, sealing is complete. In some cases a third application may be necessary.

Excess DC Copalvarnish (e.g. on enamel margins etc.) can be removed with a carbide tipped finishing burr. Restoration materials are then used in the normal way.

### Storage

Close bottle immediately after use, otherwise a change of consistency could be happened.

### Presentation

1 x 10 ml liquid

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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