DC Dentine-Composite Bond

DC DentalCentral GmbH • Owidienfeldstr. 6 • D-30559 Hannover • Germany • www.dental-central.de • E-Mail: office@dental-central.de

D Gebrauchsinformation

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GB Instruction for use

DC Dentine-Composite Bond is an adhesive with outstanding adhesion to dentin and etched enamel. It improves adhesion between the composite and tooth surface.

Indications

DC Dentine-Composite Bond is used to improve the adhesion of composites on enamel and dentin. Further, it improves the adhesion between glass-ionomer underfill (lining) cement and composite fillings („sandwich technique“).

Application

The cavity should be prepared according to the rules for adhesive filling technique. Depending on the situation, apply calcium hydroxide paste (DC Cal) or an underfill (lining) cement in deep cavities. The enamel is etched, rinsed and then dried in accordance with the etch bonding technique. The glass-ionomer cement filling should not be etched, only roughened. Once the prepared cavity is dry, 1-2 drops of light-curing DC Dentine-Composite Bond are dripped onto the mixing pad (immediately reclose the bottle) and a thin layer is applied to the tooth using a brush or foam pellet. Excess material is removed using a dry brush or a pellet. The material is then cured using halogen light (15 sec.).

Side effects

Undesirable side-effects are extremely rare if the product is prepared and used properly. Should you become aware of any undesirable side-effects, including in cases of doubt, please notify us, giving as precise a description of the attendant circumstances and symptoms as possible.

Working instructions

Ensure that the working instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the accompanying circumstances.

Contra-indications

Do not use DC Dentine-Composite Bond in the area of the pulpa.

Storage/shelf-life

3 years, store in a cool, dry place.
Protect against light!

Presentation

1 x 10 ml liquid

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

Safety indication

H315: Causes skin irritation.

H319: Causes serious eye irritation.

H335: May cause respiratory irritation.

P302 + P352:

IF ON SKIN: Wash with plenty of water

P305 + P351 + P338:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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