## D Gebrauchsinformation

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Entspricht: EN ISO 4049

## GB Instructions for use

### Light-curing flowable resin-based dental restorative material

#### Description

DC Flow is a light-curing, very fine-particle hybrid composite for the restorative therapy. DC Flow cures with light at a wavelength range of 400–500 µm (blue light).

#### Composition

The monomer matrix is composed of Bis-GMA, urethane dimethacrylate, and triethylene glycol dimethacrylate (30.7 % weight). The inorganic fillers contain mainly Ba-Al-fluorosilicate glass and SiO<sub>2</sub>. Additional contents: catalysts, stabilizers, and pigments (0.8 % weight). The total content of inorganic fillers is 68.5 % weight or 51.4 % volume. The average particle size is 0.7 µm.

#### Indication

- Anterior restorations (Classes III, IV)
- Class V restorations (cervical caries, root erosion, wedge-shaped defects)
- Small Restorations in the posterior region (Class I)
- Mini cavities, minimal invasive dentistry
- Veneering of discoloured anterior teeth
- Splinting of mobile teeth
- Preventive resin restorations in molars and premolars
- Repair of composite and ceramic veneers
- Luting of porcelain and composite veneers
- As the first layer of Class I or II restorations

#### Contraindication

- Placement of DC Flow restorations is contraindicated:
  - if a dry working field cannot be established, or if the stipulated technique cannot be applied.
  - if a patient is known to be allergic to any of the ingredients in DC Flow.

#### Side effects

In individual cases, components of DC Flow may lead to sensitization in predisposed persons. DC Flow should not be used in such cases. To avoid possible irritation of the pulp, areas close to the pulp should be protected with a suitable pulp/dentin protector.

#### Interactions

Substances containing eugenol/oil of cloves may inhibit the polymerization of composite materials. Consequently, application of such materials together with DC Flow must be avoided. Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents, and chlorhexidine.

#### Application

##### 1. Shade determination

Clean the teeth prior to shade determination. The shade is selected with the tooth still moist.

##### 2. Isolation

Appropriate isolation, best with a rubber dam, is required.

##### 3. Cavity preparation

Cavity preparation is carried out according to the requirements of the adhesive technique, i.e. protecting the tooth structure. Do not prepare sharp, internal edges and no additional undercuts in caries-free areas. The dimensions of the cavity are generally determined by the extent of the caries or the size of the old restoration. Bevel enamel edges in the anterior region. For the posterior region, only the sharp enamel edges should be rounded (finishing diamonds, 25–40 mm). Caries-free cervical defects are not prepared, only cleaned with pumice or other suitable cleaning pastes with the help of rubber cups or rotary brushlets. Subsequently, remove all residue in the cavity with water spray.

##### 4. Pulp protection / Base

Do not apply a base material when applying an enamel/dentin bonding agent. Only cover very deep areas close to the pulp with a calcium hydroxide material and subsequently use a pressure resistant cement (e.g. a glass ionomer cement). Do not cover other cavity walls, since they can be used to support the bond with an enamel/dentin adhesive.

##### 5. Matrix / Interdental wedge

Use a transparent matrix for cavities affecting the proximal area and wedge it. The use of steel matrices in the posterior region is also possible.

##### 6. Enamel and dentin conditioning

Apply 37% phosphoric acid (e.g. DC Ätzgel) on the enamel and dentin (starting at the enamel margins) and let it react for 15 secs. After that, remove phosphoric acid with copious amounts of water. Then, dry tooth surface with oil-free air. Avoid dehydration of the dentin. The etched enamel surface should have a chalky white appearance. If this is not the case, repeat the etching procedure. In case of the tooth surface being contaminated (e.g. with saliva), the etching procedure must also be repeated.

##### 7. Application of the bonding agent

Given the coordination of DC materials, we recommend using DC Adhesive bonding agent. The procedure when using DC Adhesive is described below. For further information, please refer to the Instructions for Use of the corresponding bonding agent.

Apply DC Adhesive on the conditioned tooth surface with a brush (10 secs.), wait 15 secs. and disperse excess material with oil-free air, until movement of the liquid is no longer visible.

##### 8. Application of DC Flow

Apply DC Flow in layers of max. 2 mm and adapt with suitable instruments. Polymerize each layer for 40 secs. with a curing light. Hold the light emission window as closely as possible to the surface of the restorative material.

When using a metal matrix, additionally polymerize from the buccal or lingual / palatal respectively, after removing the matrix.

##### 9. Finishing / Checking the occlusion / Polishing

Remove excess material with suitable finishers or fine diamonds after polymerization. Remove proximal excess with diamond or tungsten carbide finishers or finishing strips. Check occlusion and articulation and apply appropriate grinding corrections to prevent premature contacts or undesired articulation paths on the surface of the restorations. Use silicone polishers, polishing discs, and polishing strips to polish the restorations to a durable high gloss.

#### Additional information

1. If necessary, additional DC Flow can be directly applied to polymerized material. If the DC Flow restoration has already been polished, it must first be roughened and wetted with a bonding before fresh DC Flow is applied.
2. DC Flow should have room temperature when applied. Cool temperatures render the material difficult to extrude.

#### Warning

Unpolymerized DC Flow should not come in contact with skin, mucous membrane, or eyes. Unpolymerized DC Flow can have a slight irritating effect and may lead to a sensitization against methacrylates.

#### Storage

- Do not use DC Flow after the indicated date of expiration.
- Store packages at 2–28° C / 36–82° F.
- Close syringes immediately after use. Exposure to light causes premature polymerization.
- Shelf life: see information on syringes / packages.

Keep material out of children's reach!  
For use in dentistry only!

*This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the instructions for use. Liability cannot be accepted for damage resulting from failure to observe the instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and data constitute no warranty of attributes and are not binding.*

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