# DC Inlay-Onlaycem

DC DentalCentral GmbH • Owienfeldstr. 6 • D-30559 Hannover • Germany • www.dental-central.de • E-Mail: office@dental-central.de

## D Gebrauchsinformation

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## GB Instruction for use

DC Inlay-Onlaycem is a glass ionomer cement for attachments with exceptionally good adhesion on tooth enamel and dentine. Its outstanding features include low solubility in the mouth and low acidity.

**Components**  
Silicate powder, Polyacrylic acid.

**Fields of application**  
For attachments of crowns, bridges, inlays, onlays, pins and orthodontic bands.

**Contra indications**  
No direct application on the dentine.

### Method of use

Clean enamel, dentine and metal surfaces carefully and dry thoroughly. It is important to ensure that no traces of eugenol are left which will detract from DC Inlay-Onlaycem effectiveness. It is therefore advisable to attach temporary work with DC EugenolfreeCem eugenol-free cement. Shake powder well before use! DC Inlay-Onlaycem has to be applied at room temperature (15 – 23° C). In case of higher temperatures it is recommended to use a cooled glass-plate. Put powder and liquid on a glass-plate or on a mixing-pad and work the powder portion by portion into the liquid. Use 2 drops of liquid for 1 level measuring scoop of powder. The mixing time is approx. 30 seconds. DC Inlay-Onlaycem can be worked for approx. 1'20" minutes at room temperature and sets in the mouth within approx. 3 minutes. Higher temperatures will shorten the working time and lower temperatures prolong it. In case of bridges and crowns which are too narrow it is important to ensure that the excess of DC Inlay-Onlaycem is not too great, nor its viscosity too high, since glass ionomer cements have specific flow properties.

DC Inlay-Onlaycem has the right consistency if it can be applied to the tooth stumps and inside the crown by using a stiff brush. During application and setting time prevent contact of work with water and saliva.

### Side effects

Side effects with this medical product are extremely rare if it is properly prepared and applied. Should you encounter undesirable side effects – even in case of doubt – then please contact us and give a precise description of the circumstances and symptoms.

### Processing instructions

Ensure that the processing instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable re- actions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the circumstances.

### Supplementary indications

Rinse metal instruments with cold water immediately after use and before any cement still attached has time to set. After removal of orthodontic bands, cement residues can be removed gently and rapidly by means of an ultrasonic scaler. Close powder container immediately after use.

DIN EN ISO 9917-1 requirements are fulfilled.

**Presentation**  
24 g powder  
10 g liquid

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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