DC Knetmasse x-hard

DC DentalCentral GmbH • Owienfeldstr. 6 • D-30559 Hannover • Germany • www.dental-central.de • E-Mail: office@dental-central.de

D Gebrauchsinformation

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GB Instructions for use

Kneadable silicon for dental technology.

Silicon material for the dental technology, condensation curing.

Application: preliminary bulwarks and small models.

DC kneading paste x-hard

The silicon material will be carried out manually in according to the dosage.

Colour	DOSAGE			Mixing time	Total working time	Setting time	Hardness Shore A
	Basispaste	Activator					
		Paste	Liquid				
Blue	1 measuring spoon 12 ml = (22,4 g)	2,5 graduation = 2,5 cm 0,53 ml = (0,5 g)	10 drops 0,24 ml = (0,24 g)	30 seconds	1 minute 30 seconds	6-7 minutes	85

1 graduation = 1,0 cm (related to the mixing pad)

Results from 23° C, 50 ± 10 % rel. humidity, higher temperatures shorten, lower temperatures prolong the setting times.

Handling and safety measures:

Avoid contact with eyes, if contact should occur, rinse eyes immediately and thoroughly in clean running water and go directly to an eye specialist. Avoid getting traces of the impression mass on clothing, as its chemically bonds to clothing fibers. Close paste hardener vessels carefully after use. The preparation, usage and application of the products supplied lie exclusively in your area of responsibility since we have no opportunity of exercising control over these matters. The advice given on technical application matters verbally, in writing or through trials, then this advice is to be understood as nonbinding information; the same applies in respect of the possible protective rights of third parties. Such advice does not release you from the obligation of testing the products supplied for their suitability for the intended purpose and processes. Our liability is in every case of damage limited to the value of the materials supplied and the goods used. The proper quality of our products is guaranteed in accordance with the terms of our general conditions for sale and delivery.

Warnings:

Inhalation: Bring the person into the fresh air and treat with oxygen if necessary.

Eye contact: Avoid eye contact, if necessary, irrigate the eyes immediately with plenty of clear water and consult an ophthalmologist promptly.

Skin contact: Avoid prolonged skin contact. After prolonged contact wash the skin with soap and water. We recommend wearing suitable protecting clothing and gloves.

Swallowing: After swallowing, rinse out the mouth and drink plenty of water. Always contact a doctor.

Allergic reactions: Allergic reactions can occur on sensitive persons. In case of doubt, consult a dermatologist or allergologist. If acute allergic reactions should occur during the treatment, treatment with the product should be stopped immediately.

Storage: Do not store above 27° C.

Shelf life: see information on package.

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DC DentalCentral GmbH • Owiedenfeldstr. 6 • D-30559 Hannover