D

Gebrauchsinformation

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GB

Instructions for use

DC Silvercem is an X-ray opaque glass-ionomer cement to build-up stumps and block-out undercuts. DC Silvercem is also a filling material for Class I cavities, tunnel preparations and for milk teeth. DC Silvercem is light grey which is given a metallic gloss after polishing. DC Silvercem additionally contains a fine-grain silver that makes it X-ray visible. Adhesion to the dentin and enamel is by way of a reaction of the carboxylate groups of polycarboxylic acid with the calcium of the hydroxylapatite and collagen. This direct chemical bonding ensures that much more healthy dental material can be retained because undercutting is not necessary. After a while fluoride is released.

Preparing the cavities

As much healthy dental material as possible should be retained because undercutting is not necessary to achieve better adhesion. The edge should not be trimmed down to thickness of less than 0.5 mm.

Pulpa protection

Rinse the prepared cavities thoroughly with water and dry in a stream of oil-free air. Use calcium hydroxide preparations (e.g. DC Cal) as pulp-capping for very deep cavities.

Preparing the dentin

Clean the freshly prepared dentin surface to remove plaque, rinse thoroughly with water and dry with a stream of air. Avoid over-drying!

Mixing DC Silvercem

Use one level dosing scoop with one full drop of liquid (without bubbles; hold the bottle vertically downwards and apply light pressure to squeeze out one drop). First mix only half the amount of powder, then continue with the other half of the remaining powder until the required consistency is achieved. Use only a small area of the mixing block in order to minimize water evaporation. Powder/water mixing ratio (w/w): 2.8 : 1

Application of DC Silvercem

Place the mixed material is directly in the cavity where it is compacted. If necessary, shape the surface with a matching matrix, or cover it with an ionomer varnish (e.g. DC Copalvarnish). DC Silvercem is workable for 2 minutes (incl. mixing). The material should not be handled any more when hardening sets in. The hardening time is 3 - 3.5 minutes.

Trimming

When use as a core build-up material, excess material can be removed after 5 minutes with a Vaseline-coated high-grade corundum grinder or with flexible grinding wheels. An impression can then be immediately made. When used as a filling material polishing can only commence after 8 minutes. Here, too use Vaseline-coated grinding wheels or grindstones. Coat the surface with a protective varnish to protect the ionomer cement from water during the first 24 hours.

WARNING: Do not use waterspray!

Please Note

- WARNING: Do not allow non-bonded material, dental cleaner or ionomer varnish to come into contact with the eyes. Should this nevertheless inadvertently happen, then rinse immediately with plenty of water.
- Instruments must be immediately cleaned under flowing water after they have been used!

Side-effects

Side-effects with this medical product are extremely rare if it is properly prepared and applied. Should you encounter undesirable side-effects – even in case of doubt – then please contact us and give a precise description of the circumstances and symptoms.

Processing instructions

Ensure that the processing instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the circumstances.

Composition

DC Silvercem contains ionomer glass, silver alloy and polyacrylic acid.

Storage

Do not use after expiry of the use-by date. Opened packs can be kept for 6 months if they are kept dry and stored at room temperature.

Packaging

12 g Powder, 7 g Liquid + dosing spoon

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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