DC DENTAL CENTRAL

DC Temporaryfill

DC DentalCentral GmbH • Owidenfeldstr. 6 • D-30559 Hannover • Germany • www.dental-central.de • E-Mail: office@dental-central.de

D

Gebrauchsinformation

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GB

Instructions for use

DC Temporaryfill is a proven, ready-to-use close-up-material. It guarantees reliable sealing due to the following properties:

- rapid curing in the cavity
- strong adhesion to dentin
- outstanding marginal seal
- resistant to chewing pressure

DC Temporaryfill damages neither the pulpa nor the gums.

Type of application

Use a filling instrument to apply the required quantity of DC Temporaryfill in the moist cavity. Curing begins after a few minutes. We recommend protecting the exposed location for 15 minutes. Deep cavities do not have to be completely filled, which makes subsequent removal easier. When sealing inlays, as far as is possible, ensure pressure-free application. A pointed, short and strong probe is best suited for removing the set DC Temporaryfill. Insert it between the dentin and filling.

Side effects

Undesirable side-effects are extremely rare if the product is prepared and used properly. Local dysaesthesia (e.g. taste irritations or irritations of the oral mucosa) can however, not be completely precluded. Should you become aware of any undesirable sideeffects, including in cases of doubt, please notify us, giving as precise a description of the attendant circumstances and symptoms as possible.

Working instructions / precautionary measures

Ensure that the working instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the accompanying circumstances.

Storage / shelf-life

Always close container immediately after removing material. Shelf-life of the closed packaging: See label.
The preparation can be stored for at least 6 months after being opened for the first time. Do not store above 25° C. Do not use after expiry of the use-by date.

Type of packaging

2 glass jars with 30.0 g each

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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