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## GB User information and reprocessing

The following instructions provide best results in preparations through careful handling of the diamond instruments and their application. Consideration of the teeth as well as perfect hygienic conditions are the main concern.

- Instruments must be sterilized before first and any further usage in an autoclave sterilizer with a sterilization pressure of 1-2 bar, a min./max. temperature of 120° C / 150° C and a min. sterilization time of 25 minutes.
- Instruments / burs should be cleaned in a „cleaning solution“ first after use. (e.g. DC Bohrerbad NF). See manufacturers instructions for details.
- Instruments / burs can be brushed to remove material sticking to the diamond layer.
- Dental diamond instruments are made of stainless materials. However, they must not get in contact with acids.
- Trays or original packing should be used for storing.
- Instruments are to be selected by the head diameter to prevent from damaging neighbour teeth or tissue.
- Rounded shapes should be favoured.
- Instruments with coarse grit are not always indicated; usually they require a finish grinding operation.

- The shafts have to be secured properly and checked for minimum run out before use.
- The burs should be set to the working speed according to the original packing. It should be set before applying the bur to the object.
- Spray cooling with a minimum flow of 50 ml/min. must be provided.
- Additional cooling is required as burs are longer than 21 mm or heads are bigger than 2 mm being used.
- Forces should not exceed 2 N (appr. 2000 p). Higher forces do not result in a quicker material removal, but increase the thermal load on the tooth.
- Worn or damaged burs must be replaced immediately.

Our dental diamond tools are manufactured from high-quality components providing a long life time. However we recommend not to use the instruments after an expiration time of 10 years.