Polierer von DC DentalCentral

DC DentalCentral GmbH • Carl-Zeiss-Str. 2 • D-22946 Trittau • Germany • www.dental-central.de • E-Mail: office@dental-central.de

Stand 01/2012
DCWA007

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GB Instructions for use and reprocessing

These instructions are applicable to the following DC products:

- DC prophyl brushes, coloured
- DC Tuft Brush
- DC OneStepBrush Gold
- DC rubber cup (different designs)
- DC one knot brush
- DC Mandrell
- DC COMP-polisher

These rotary instruments are supplied mechanically clean but are not sterile. They should therefore be sterilized before first use respectively before single use. The reusable devices have to be reprocessed before each use.

INSTRUCTIONS

Workplace

Remove surface contamination with a paper towel.

Storage and transport

No specific requirements. It is recommended to process the instrument as soon as possible after using it.

Cleaning preparations

See disinfection and sterilisation. Disassembly is not necessary.

Cleaning: Manual

Equipment: cleaning agent (e.g. DC Instrumentendbad), soft brush, running water.

Procedure:

1. Wash off surface contamination on the instrument (temp. < 30° C)
2. Apply cleaning solution with a soft brush on all surfaces. Use a suitable brush for cleaning cannulizations and blind holes so that all places will be reached.
3. Hold the instrument under running water for 3 minutes. The running water must rinse out the cannulizations, and the blind holes must be filled with water and emptied several times.
4. After cleaning inspect the instruments to ensure that all contamination has been removed. Repeat the cleaning process if necessary

Cleaning: Automatic

Equipment: cleaning/disinfection machine, cleaning agent (e.g. DC Instrumentenbad)

Procedure:

1. Load the instruments into the machine.
2. Set on standard cycle, wash for at least 5 minutes and rinse for 3 minutes.
3. When removing the instruments, check cannulizations, blind holes, etc. for visible contamination. If necessary, repeat cycle or clean manually.

Desinfection

Disinfectant, can be used according to the instructions on the label.

When cleaning automatically, the equipment can be rinsed at the end for 10 minutes at a temperature of 80° C.

Maintenance

Identify and replace blunt or damaged equipment.

Checking and performance test

All instruments: visual inspection for damage and wear and tear. Corners and edges should not show notches and should be even. Check long and narrow instruments (particularly rotation instruments) for distortions. If instruments belong to a larger system, it should be checked together with the corresponding elements.

Limits to processing

Frequent processing impacts the instruments only slightly. The end of the product lifecycle is normally determined by wear and tear and/or damage through use.

Sterilisation

Vacuum autoclave, at least 20 minutes at 134° C, or according to RKI-directive, not exceeding 140 °C. The instruments are not suitable for sterilisation in chemiklave or a hot air disinfectant.

Storage

No specific requirements.

Additional information

The autoclave manufacturer's instructions must be followed. When sterilising several instruments in an autoclave, the indicated maximum load prescribed by the manufacturer may not be exceeded.

Used instruments should be considered as contaminated and appropriate handling precautions should be taken during reprocessing. Globes, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross contamination risks from the patient.

Additional information to the individual instrument:

Product	Additional information to the individual instrument:
DC prophyl brushes, coloured (Single use)	Prolonged storage in disinfectant solutions may result in corrosion and should be avoided. After cleaning dry the instrument using paper towelling or dry heat not exceeding 140° C.
DC Tuft Brush (Single use)	Prolonged storage in disinfectant solutions may result in corrosion and should be avoided. After cleaning dry the instrument using paper towelling or dry heat not exceeding 140° C.
DC OneStepBrush Gold	Prolonged storage in disinfectant solutions may result in corrosion and should be avoided. After cleaning dry the instrument using paper towelling or dry heat not exceeding 140° C. Do not reprocess DC OneStepBrush more than 20 times!
DC rubber cup (different designs) (Single use)	Prolonged storage in disinfectant solutions may result in corrosion and should be avoided. After cleaning dry the instrument using paper towelling or dry heat not exceeding 140° C.
DC one knot brush (Single use)	Prolonged storage in disinfectant solutions may result in corrosion and should be avoided. After cleaning dry the instrument using paper towelling or dry heat not exceeding 140° C.
DC mandrell	Prolonged storage in disinfectant solutions may result in corrosion and should be avoided. After cleaning dry the instrument using paper towelling or dry heat not exceeding 140° C.
DC COMP-polisher	Non-compliance with the maximum allowable RPMs will lead to an elevated safety risk. Insufficient water cooling may injure the teeth and the contiguous tissue irreversibly.

RPM instructions

The maximum allowable RPMs must not be exceeded. The recommended and maximum allowable RPMs vary from product to product. To check the recommended and maximum allowable RPMs please consult our current catalogues or the packaging. If the maximum allowable RPMs are exceeded, this may cause vibrations, which can destroy the polisher, bend the shaft and/or break the instrument, i.e. it may lead to an increased safety risk for users, patients or third parties. Compliance with the recommended speed range will lead to the best work results.

The manufacturer has confirmed that the above-mentioned instructions are suitable for preparing the re-use of an instrument. The person processing the instrument bears the responsibility that the actual processing with the used equipment, materials and employees in the processing installation will produce the desired results. Normally, validations and routine monitoring of the process are necessary for that. In addition, each deviation from these instructions by the processing person should be evaluated diligently in terms of efficiency and potential negative consequences.